

WISHOWER

QUICK INSTRUCTION GUIDE



MANDATORY ACCESSORIES FOR THE HYGIENIC CARE

(PLEASE REFER TO THE ACCESSORIES SHEET FOR MORE DETAILS)



QUICK INSTRUCTION GUIDE

Step by step instructions

1. Plug the device using the power cord near a water access (bathroom or bathing area).



2. Please complete the desired settings: using the touch screen on the control panel, select "**Option**" to choose the language, time, 12h or 24h format of the time, Fahrenheit, or Celsius degrees, black or white theme of the digital screen and adjust the brightness.



3. Pour **warm water** into the clean water tank opening until all four (4) water level bars appear on the touch screen. To minimize water heating time, **pay attention to the temperature variation while filling** the clean water tank and adjust if needed. When the 4th bar appears, it means it has reached its full capacity.



- **Caution:** Please exercise caution and ensure that the device is never left unattended during the filling process to prevent any potential overflow. When filling, cease adding water as soon as you observe the 4th level indicator, represented by the fourth light bar on the screen, **to avoid exceeding** the maximum capacity of the clean water tank.



- **Caution:** Please be aware that for safety reasons, the internal safety mechanism will prevent the mobile shower from being activated if the water temperature **reaches or exceeds 40°C (104°F)**. The digital touch screen will show a visual warning (clean water tank in red), accompanied by an audible alarm to alert the operator. To address this, you can either add cooler water or patiently wait until the temperature decreases.

4. Unplug and bring the device to the patient's room.

5. Place the device facing the patient's bed and plug the power cord once again. The digital touchscreen will turn back on automatically.



6. Engage both foot brakes on the front wheels for stability and security.



7. Select your preferred water temperature from the options available on the digital touch screen: either 37°C (98.6°F) or 38°C (100.4°F). Once you've made your selection, the water heating process will commence automatically and complete within a few minutes, depending on the initial water temperature.



8. Installation of the patient in bed:

- **If the patient is not in bed:** Secure the 4 clamps on the footboard and headboard of the bed. Install the bed size towel on the bed. Place the waterproof bed cover atop the bed size towel and insert the clamps in the eyelets of the bed cover.
- **If the patient is in bed:** Secure the 4 clamps on the footboard and headboard of the bed. Turn the patient to the side. Slide the bed size towel under him, then slide the waterproof bed cover over the bed size towel. Use the waterproof bed cover's eyelets to secure 2 clamps on this side of the bed. Repeat this step for the other side of the bed.



9. Undress the patient. It is recommended to cover the patient with a towel to protect his or her privacy. That towel can easily be moved during the washing.

10. On the digital board, press "**Shower**" and "**Suction**" functions to start the shower in bed (blue= idle, green= activated).



11. Press the black button of the shower head for the water flow and test its flow and temperature away from the patient.

12. Simultaneously suction the water into the waterproof bed cover with the suction hose.



13. Once finished, bring the shower hose and suction hose back to the device and turn it off by unplugging it.

14. Turn the patient to the side to remove the soiled waterproof bed cover by folding it under him. Repeat this step for the other side.



15. Set aside the soiled waterproof bed cover.

16. Wrap the patient in the bed size towel to dry completely.

17. Dress the patient (usual procedure of patient's grooming).

18. Bring the reusable waterproof bed cover and the bed size towel to the laundry room.

For washing instructions and recommended maintenance for both the reusable waterproof bed cover and bed size towel, please refer to the "*Wishower accessories*" section of the *Wishower User Manual*. We also offer single use accessories. For further details, please consult with your representative.

Post hygienic care procedures

Please proceed to the *Disinfection process* following the step-by-step instructions along with the *Infection prevention protocol and best practices*.