

WISHOWER

PREVENTIVE MAINTENANCE GUIDE



WISHOWER



CONTACT INFORMATION

LIOMAR Medical Inc.

Office address

211-9250 Park Avenue
Montreal, Quebec, Canada H2N 1Z2

Office phone number

+1 (450) 987-1127

Administration email address

Info@liomarmedical.com

Website

www.liomarmedical.com

Health Canada Medical Device

Establishment License No. 16917


TABLE OF CONTENT

PERIODIC MAINTENANCE	4
Device exterior components	4
Visual and physical inspection.....	5
External removable filter	5
Wheels and foot brakes.....	5
ANNUAL PREVENTIVE MAINTENANCE	5
Tests and controls.....	6
TECHNICAL TROUBLES	6

PERIODIC MAINTENANCE

It is recommended to do periodic preventive maintenance controls for the mobile shower Wishower™. A quick general inspection of the exterior components can ensure the proper functioning of the device and prevent potential damages if some components require immediate attention.

Device exterior components

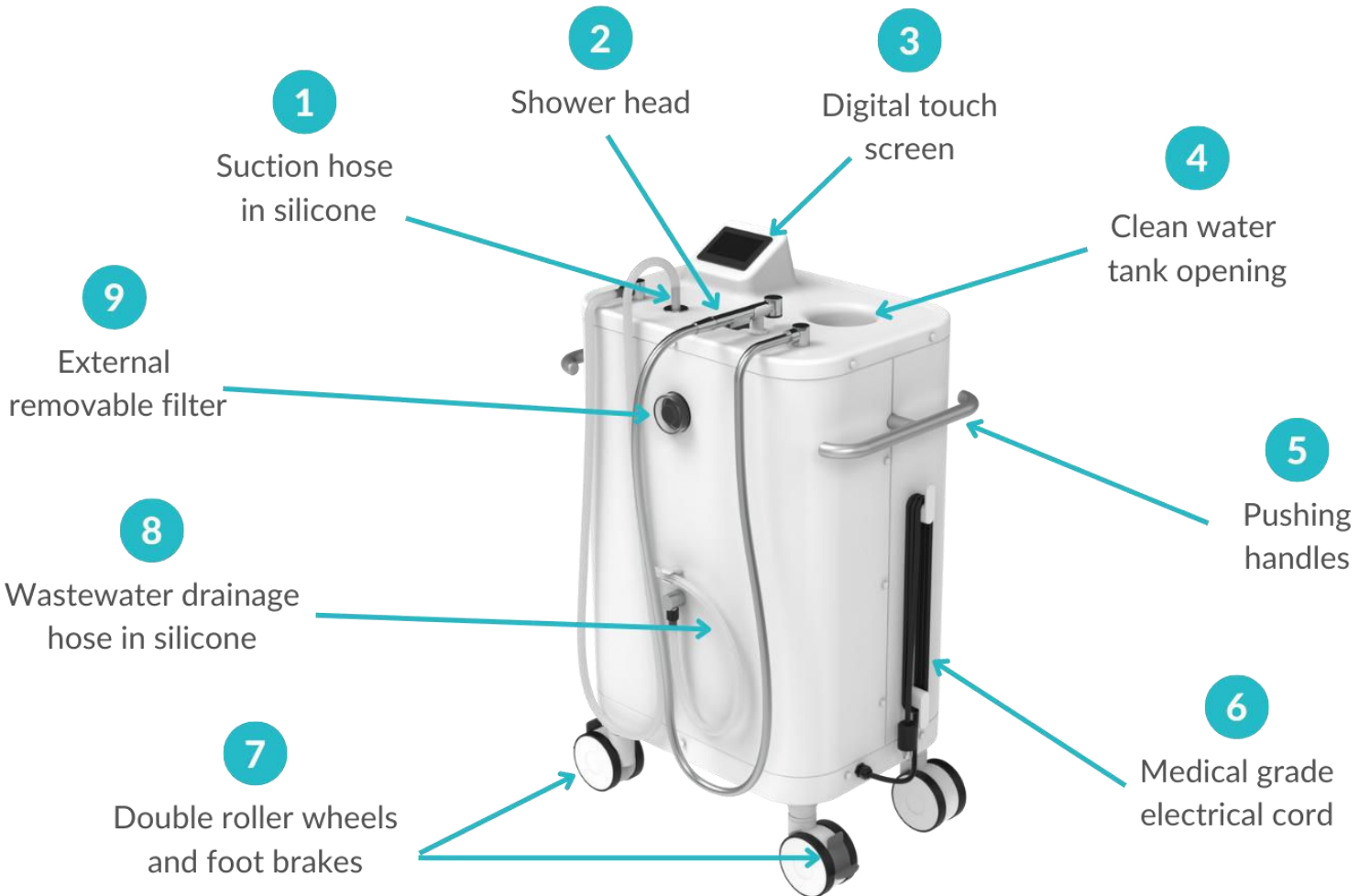

Visual and physical inspection

It is important to inspect visually and physically the external components of the Wishower™ to ensure proper functioning of the device. Verify the general condition of the following components to see if there are any damages, cracks, visible obstructions or accumulated dirt and residues :

- The plastic exterior components of the mobile shower
- The external removable filter and its protective cap
- The four (4) double roller wheels and the two (2) side-by-side foot brakes on wheels (mobile shower is easily moved and the breaks are stable)
- All 3 hoses (shower, suction, and wastewater drainage hose)
- All reusable accessories
- Accessories inventory

If you notice any irregularities or problems, please ask your Wishower™ representative for support or technical advice. You can also refer to the *User Manual* and this *Preventive Maintenance Guide* for more details on how to use the product.

External removable filter

We recommend cleaning out the filter every 4 to 5 hygienic cares, or as needed.

Wheels and foot brakes

If the steering of the device becomes harder than usual or the wheels make a disturbing sound while rolling, you may need to check for any obstruction, damage, or loose wheels. If you have any question or need to replace wheels, please contact your Wishower™ representative.

ANNUAL PREVENTIVE MAINTENANCE

The annual maintenance, particularly important if the device is being used daily, should be carried out either by qualified personnel who have received LIOMAR Medical training on the device or personnel specifically assigned to medical equipment technical maintenance and issues.

Depending on the nature of the technical problems, the facility's technicians can be trained on request to carry out certain preventive and curative interventions with Wishower™. The information and costs for technical support can be provided upon request.

It is recommended to annually check the condition of all the external components of the device and replace them if necessary.

Tests and controls

It is suggested to do these following tests during the annual inspection of the device.

- Fill the clean water tank with hot added water to see if the **heating elements**, the **sensors**, and **both the visual and audible alarms** still function properly when the water's temperature exceeds the 40°C (or 104°F).
- Test the **shower head** by activating the shower function on the digital touch screen and then pressing and releasing the regulator button for the water stream. The shower head should have a comfortable and constant jet pressure.
- Test the **suction hose** and **suction tip** by activating the suction function on the digital touch screen and then suctioning some water.
- Test the **water evacuation system** by starting the decontamination process in order to evacuate grey waters. Then, you can either proceed with the decontamination process, or cancel and go back to the main menu.

TECHNICAL TROUBLES

- If you encounter any technical trouble or detect a problem during your verification and maintenance, please follow LIOMAR Medical's approved repairing procedures, or contact your Wishower™ representative to obtain *adequate assistance. If a component breaks or gets damaged following the maintenance, we can provide you with a quote according to your needs.
- For any other question, please contact your Wishower™ representative or contact directly LIOMAR Medical.

* Note that LIOMAR Medical can't be held responsible for any damages caused by an inadequate repair that is not made by a LIOMAR Medical authorized representative. Only the replacement components provided by LIOMAR Medical will be honored for warranty purposes.